

ReTurnBelt

SystemRoMedic



Max: 150 kg/330 lbs

The ReTurnBelt is used for raising from a seated to a standing position, and for support in the standing position in combination with the use of ReTurn transfer platform. The ReTurnBelt has a more rigid lower section for easy application and an elastic upper section for a better and more comfortable fit. The ReTurnBelt has several gripping alternatives and a locking strap for extra support and safety in connection with the use of the ReTurn platform.

ReTurnBelt are part of a series of different transfer assistive devices. The choice of model and material is determined by the patient's functional capacity and the situations in which the product is to be used.

RoMedic's products together form the SystemRoMedic concept, which takes a holistic approach to patient transfers. SystemRoMedic has four categories of transfers: transfer, positioning, support and lifting.



Functional inspection

Visual inspection

Check the condition and function of the product regularly, and always after laundering. Check to ensure that the fabric is not worn, damaged or bleached. If there are signs of wear, the product must be discarded.




Always read the instructions


Read instructions for all assistive devices used in connection with transfers.
To prevent discomfort, exercise care and caution when using the product.



Functional testing of ReTurnBelt

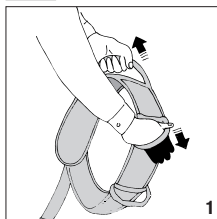


Pull hard on the handles in different directions (1).

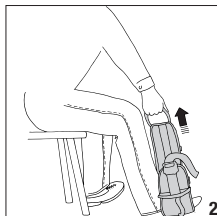


Fasten the belt, place one foot in it (2), and then pull on one of the handles. Repeat the procedure with all handles.


Placement of ReTurnBelt

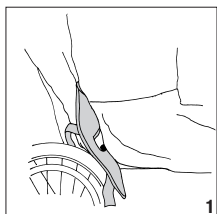


1. Fit the ReTurnBelt to the patient with the elastic section facing upward and the rigid, lower section downward, as shown in the illustration on the product label.

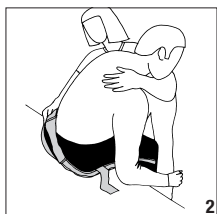

2. Lean the patient forward or to one side and guide the lower section of the belt down towards the 'sit bones' (lowest bony prominences or ischial tuberosities).

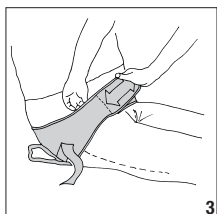


3. Secure the belt with the Velcro straps. The entire length of the front Velcro strap must be secured to the soft Velcro strap.

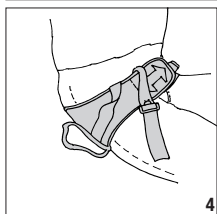


4. The locking strap can be temporarily folded under the edge of the belt, so that it is accessible when the patient has risen to a standing position. The locking strap is then looped around the ReTurn for added security while the patient is standing.

Using the ReTurnBelt

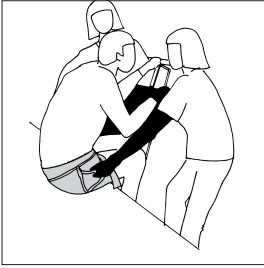


1

Brake the wheels on the ReTurn. The patient then leans forward and grasps the handles on the ReTurn. The caregivers then grasp the handles on the ReTurnBelt.

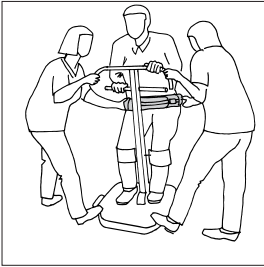


2

The caregivers then help the patient to rise to a standing position.

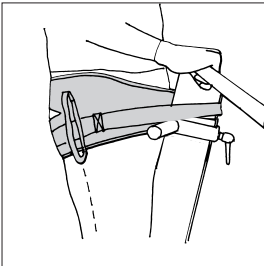

3

Ensure that the patient is standing in a secure position facing the ReTurn. Place the locking strap around the ReTurn in a suitable manner (depending on which ReTurn model is being used).

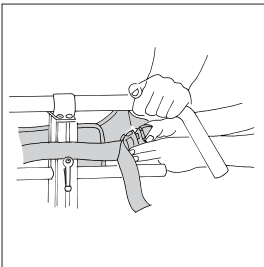

4

Secure the locking clasp. A click is heard when the clasp locks into place. Now, transfer the patient with the ReTurn. Plan the setting so as to minimize transfer time.

Sitting down

Brake the wheels on the ReTurn. Lean the patient forward, so that the locking strap is loosened. Undo the clasp and remove the strap from the ReTurn. Fold it under the edge of the belt, so that it does catch on anything. Help the patient into a seated position with the help of the ReTurnBelt.

Using ReTurnBelt without ReTurn

The ReTurnBelt can also be used to advantage even without ReTurn, for example, for gait training. Make sure you use the right size and secure the both the Velcro straps and the clasp.

Material:

Polyamid, nylon

Care of the product:

Read the product label.



Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

Fasten the Velcro straps and clasp before laundering. Clean the Velcro straps to remove dust and dirt, so that they retain their adhesiveness.

Size:

Size		XXS	XS	S	M	L	XL
Rec.	lbs	-33	33-66	55-121	110-165	154-242	230-
patient	kg	-15	15-30	25-55	50-75	70-110	105-
weight							

6031	XXS	88 cm
6032	XS	100 cm
6033	S	115 cm
6034	M	150 cm
6035	L	155 cm
6036	XL	180 cm

When in doubt, contact your RoMedic representative for product trials, advice or information.

If you require additional information, contact your transfer specialist or our representative, or visit www.romedic.com. Make sure you always have the most recent version of the manual. Manuals are available for downloading from our website; <http://www.romedic.com>

RoMedic AB

Box 640

SE-175 27 Järfälla

SWEDEN

Tel: +46 (0)8-594 773 33

Fax: +46 (0)8-594 773 44

Internet: www.romedic.com

E-mail: office@romedic.com

RoMedic®
A company in the Handicare Group