


BetterLiving Community Bed – Grab Bar

SET - UP AND OPERATION

Grab rail is designed to assist patients getting into and out of bed. The Grab Bar is supplied with two Hand Screw Knob for attaching to bed.

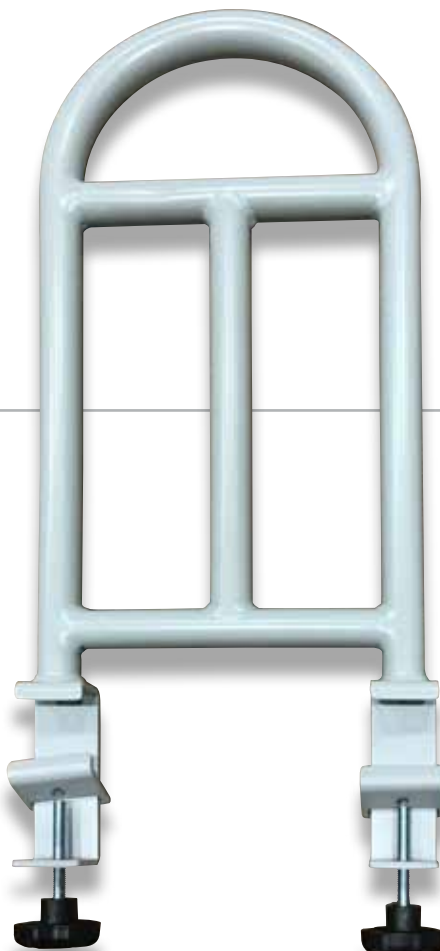

⚠ Please read these instructions before use

⚠ This accessory is designed for use with the BetterLiving Community Bed only.

Only approved bed accessories listed in the BetterLiving Community Bed Parts List should be used with the bed unless otherwise specified by Novis Healthcare.

Attach to Bed


- 1 Ensure Hand screw knobs are loosened, but not removed.
- 2 Tilt the top of the Grab Bar forward over the bed and slide the top of the securing clip over side of the bed frame
- 3 Straighten the Grab Bar to allow the bottom of the clip to slide under the bed frame
- 4 Tighten the two Hand Screw Knobs

⚠ When attaching a Grab Bar, the gap between Grab Bar and bed end must be less than 60 mm or more than 318 mm to comply with entrapment guidelines.

⚠ Ensure a risk assessment is completed first before attaching any Grab Bar to the bed.

Remove from Bed

Loosen the two Hand Screw Knobs and tilt the top of Grab Bar away from the bed and slide the securing clips off bed frame

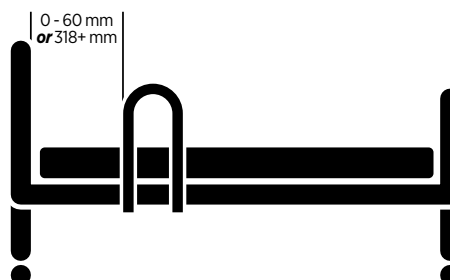

Regular maintenance

Visual inspection for obvious wear and tear of bar and components. Ensure Bar is secure by tightening Hand Screw Knobs.

Replace immediately if you see evidence of tears, cracks or other general wear.

Care & cleaning

Hand clean using hand spray and wipe methods only. Novis recommends use of officially recognized disinfectants and mild cleaning solutions.

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or the manufacturer for a list of available spare parts.


Specifications

CODE	BE9002
PRODUCT WEIGHT	2.0 kg
DIMENSIONS	W 265 x H 630 x D 55 mm
CONFORMS TO	ARTG176883
MADE IN VIETNAM	

BetterLiving Community Bed – Side Rail, Full Length

SET - UP AND OPERATION

Side rails are designed to secure patients in bed. They are easily raised and lowered. Full Length Side Rails are supplied as a pair.

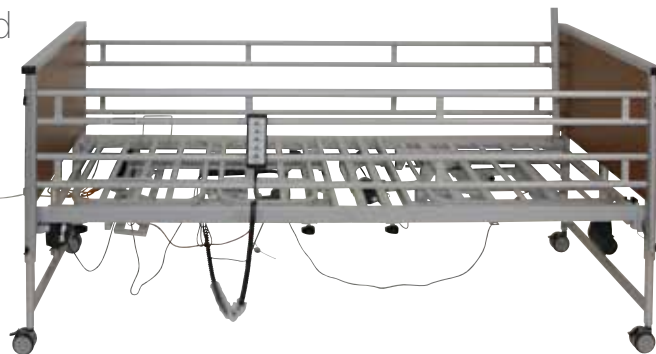

⚠ Please read these instructions before use.

⚠ This accessory is designed for use with the BetterLiving Community Bed only.

Only approved bed accessories listed in the BetterLiving Community Bed Parts List should be used with the bed unless otherwise specified by Novis Healthcare.

Product inspection

Each side rail is supplied as two components joined by a fine loop of wire (to facilitate compacting when lowering):

Protruding white plastic

Connecting wire



Attach to Bed

1 Raise the bed to its full height and ensure bed length is not extended.

2 Loosen (but do not remove) the Hand Screw Knobs at the bottom of Side Rail track on the Community Bed, at both ends of the bed.



3 Position both ends of the Side Rail with the protruding white plastic end plate pointing upwards at entrance of the Side Rail tracks.

Check Side Rail fixing wire is positioned securely in the groove of white plastic end plates before inserting into Side Rail track.



⚠ Fixing wire will be damaged if not correctly secured in the grooves.

4 Slide both ends of the Side Rail slowly vertically up in the Side Rail track – to the very top – until an audible ‘click’ is heard at both ends.

Two people may be necessary, as rail needs to be inserted into the track completely vertically.


Side Rail is now securely locked in position at its full ‘in use’ height.

5 Tighten two Hand Screw Knobs at the bottom of Side Rail track on the Community Bed – this ensures Side Rail will be held securely when it is dropped down to its lowest position.



6 Repeat the procedure above for the other side rail.

⚠ Ensure distance from top of uncompressed mattress and top of the side rail is at least 220 mm.

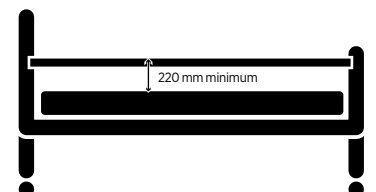

⚠ Check side rails are securely fitted before use.

⚠ All hand screw knobs at bottom of Side Rail track MUST be tightened whenever a side rail is attached to the bed. NEVER loosen hand screw knobs at bottom of side rail tracks unless installing or removing the Side Rail.

Loose hand screw knobs at bottom of Side Rail can result in serious injury and damage.

BetterLiving Community Bed – Side Rail, Full Length

SET - UP AND OPERATION

To Use

To lower the Side Rail, gently pull both Hand Knob (head end and foot end) at top of the Side Rail track.

To raise the Side Rail slowly pull up the Side Rail until an audible 'click' is heard at both ends



Side Rail Extension

Side rails have built-in length extension feature, and will automatically extend when the Community Bed extension is utilised.

No manual adjustment is necessary.



To remove from bed

- 1 Raise bed to its full height then lower Side Rail to its lowest position.
- 2 With one hand or someone holding the Side Rail, loosen two Hand Screw Knobs at bottom of Side Rail track.
- 3 Slowly let the Side Rail slide vertically down out of Side Rail track.

Regular maintenance

Daily checks are recommended for the correct and safe function of the Side Rail, including but not limited to:

- ◊ Check for signs of overall damage and locking mechanisms
- ◊ Check hand screw knobs are fastened and secure
- ◊ Audible "click" is heard when raising side rails to its full "in use" position
- ◊ Check side rails are correctly secured in the "in use" position by applying moderate downward force. If there are excessive movements when raised to its full "in use" position, DO NOT use the side rail and report the fault immediately.

Replace immediately if you see evidence of tears, cracks or other general wear.

Care & cleaning

Hand clean using hand spray and wipe methods only. Novis recommends use of officially recognized disinfectants and mild cleaning solutions.

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or the manufacturer for a list of available spare parts.

Specifications

CODE	BE6004
PRODUCT WEIGHT	7 kg each
DIMENSIONS	w 410 x h 2080 mm
CONFORMS TO	ARTG176883
MADE IN VIETNAM	

BetterLiving Community Bed – Side Rail, Three Quarter Length

SET - UP AND OPERATION

Side rails are designed to secure patients in bed. They are easily raised and lowered. Three Quarter Length Side Rails are supplied as a pair.



⚠ Please read these instructions before use.

⚠ This accessory is designed for use with the BetterLiving Community Bed only.


Only approved bed accessories listed in the BetterLiving Community Bed Parts List should be used with the bed unless otherwise specified by Novis Healthcare.

Attach to Bed

- 1 Before attaching the Side Rails to bed frame, loosen the two hand screw knobs on the Side Rail.
- 2 Position the Side Rail over the bed frame and tilt forward and clip over the horizontal support frame.




⚠ Ensure either end of the side rail is less than 60 mm or more than 318 mm from the bed end to minimise risk of entrapment.

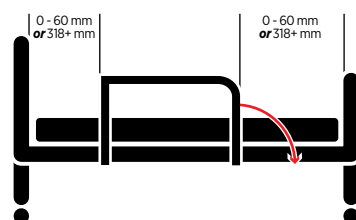

- 3 Slide the bottom fixing under the bed platform by straightening up the side rail
- 4 Tighten the two Hand Screw Knobs (to secure the Side Rails to under the bed frame)
- 5 Repeat the procedure for the other side rail.



⚠ Ensure distance from top of uncompressed mattress and top of the side rail is at least 220 mm.

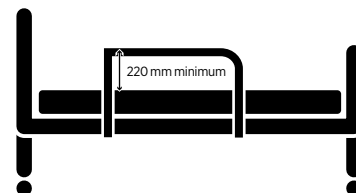

⚠ Check side rails are securely fitted before use.

⚠ All hand screw knobs at bottom of Side Rail track MUST be tightened whenever a side rail is attached to the bed. NEVER loosen hand screw knobs at bottom of side rails unless installing or removing the Side Rail.

Loose hand screw knobs at bottom of Side Rail can result in serious injury and damage.

To Use

To lower the Side Rail, gently pull the black round knob located near the foot end. Push side rail towards the Foot end

To raise the Side Rail, slowly pull up the Side Rail until an audible 'click' is heard (of the Lowering Knob clicking into position).

To remove from bed

- 1 To remove the Side Rail from the bed, loosen the two Hand Screw Knobs (under the bed frame)
- 2 Gently lift side rail and pull away from the bed



BetterLiving Community Bed – Side Rail, Three Quarter Length

SET - UP A N D O P E R A T I O N

Regular maintenance

Daily checks are recommended for the correct and safe function of the Side Rail, including but not limited to:

- Check for signs of overall damage and locking mechanisms
- Check hand screw knobs are fastened and secure
- Audible “click” is heard when raising side rails to its full “in use” position
- Check side rails are correctly secured in the “in use” position by applying moderate downward force. If there are excessive movements when raised to its full “in use” position, DO NOT use the side rail and report the fault immediately.

Replace immediately if you see evidence of tears, cracks or other general wear.

Care & cleaning

Hand clean using hand spray and wipe methods only. Novis recommends use of officially recognized disinfectants and mild cleaning solutions.

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or the manufacturer for a list of available spare parts.

Specifications

CODE	BE6003
PRODUCT WEIGHT	8 kg each
DIMENSIONS (INSTALLED, IN THE 'UP' POSITION)	434 x 1235 mm
CONFORMS TO	ARTG176883
MADE IN VIETNAM	



Pressure care and patient handling specialists

novis.com.au / 1300 738 885

JAN
2018 ©

BetterLiving Community Bed – Self Help Pole

SET - UP AND OPERATION



Self Help Pole is designed to help patient position themselves in a bed. It is supplied unassembled, in two sections with an adjustable length strap and a plastic handle bar.

- ⚠ Please read these instructions before use.
- ⚠ Safe Working Load of 75 kg
- ⚠ This accessory is designed for use with the BetterLiving Community Bed only.

Only approved bed accessories listed in the BetterLiving Community Bed Parts List should be used with the bed unless otherwise specified by Novis Healthcare.

Assemble Product

- 1 To assemble, attach two sections of the pole by inserting open ends of each section together.
- 2 Align the locking button with the corresponding hole and push in the locking button in as two ends are pushed together
- 3 An audible click can be heard and the locking button head is visible through the hole.

To disassemble, depress the locking button while pulling two sections of the pole away from each other.



Regular maintenance

Visual inspection for obvious wear and tear of bar and components.

Replace immediately if you see evidence of tears, cracks or other general wear.

Care & cleaning

Hand clean using hand spray and wipe methods only. Novis recommends use of officially recognized disinfectants and mild cleaning solutions.

Attach to Bed

Remove or move mattress to locate Self Help Pole bracket

- 1 To attach the pole to the bed, align the notch at base section of the pole to groove in the black plastic sleeve at head end of the bed.
 - 2 With the notch and groove aligned, insert the pole with a direct downward vertical force. Check the notch is in contact with the bottom of the groove.
- ⚠ Note, there are two notches on bed frame, depending on where the pole needs to be positioned for patient usage. Once in a notch, pole will not 'rotate'.

Strap may be adjusted to suit patient comfort



To Remove

To remove Self Help Pole, pull vertically upwards out of the the black plastic sleeve in the bed frame.

- ⚠ Do not pull at an angle or rotate the pole.



Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or the manufacturer for a list of available spare parts.

Specifications

CODE	BE7003
MAXIMUM USER WEIGHT	75 kg
PRODUCT WEIGHT	4.5 kg
HEIGHT	1400 mm
CONFORMS TO	ARTG176883
MADE IN VIETNAM	

BetterLiving Community Bed – Transport Bracket

SET - UP AND OPERATION

The Transport Bracket is designed for the practical, compact and secure transportation and storage the bed and accessories

Bracket is supplied as a non-identical pair. Bracket can be attached under Community Bed frame for storage when not in use.

Two people will be required to utilise transport brackets



⚠ Please read these instructions before use.

⚠ This accessory is designed for use with the BetterLiving Community Bed only.

Only approved bed accessories listed in the BetterLiving Community Bed Parts List should be used with the bed unless otherwise specified by Novis Healthcare.

1 Unassemble bed, as described in the Community Bed User Guide

2 Lock the bed end castors

3 Position the brackets between upright Bed Ends.

Hook the Transport Bracket onto both Bed Ends to hold them together.

Ensure the labels on the transport Brackets are facing out



4 Slide all 4 locking pins fully through Transport Bracket hole to hold Bed Ends securely in position.

⚠ Ensure all 4 castors are locked to prevent accidental movement and all 4 locking pins are fully inserted before continuing to next step.



5 Secure all loose cables with cable ties or equivalent.



6 Place Upper Frame Section into position on the Transport Bracket (inserting the Bracket into Joining locations).

When correctly positioned, control box and knee bend actuator should face inwards, and Foot symbol label should be facing up.



7 Place Lower Frame Section into position on the Transport Bracket.

When correctly positioned, backrest actuator should face inwards, and the Head symbol label should be facing up.



8 Slide two Connection Rods onto allocated space on the Transport Bracket



BetterLiving Community Bed – Transport Bracket

SET - UP AND OPERATION

- 9 Place Mattress retainers in slots that are now at the top of the Bed Frames



- ⚠ Disengage castors before attempting to move

Accessory Storage

Self help pole can be disassembled transported in two pieces, on each side of the Transport Bracket.




Slot onto bracket with components 'sandwiched' between Bed Frames

Transport Bracket Storage

When Transport Bracket is not in use, it can be stored under the Backrest in the upper section of mattress platform.

Slot the upper hook into knob in the bed frame. Securing it with Hand Screw Knob supplied

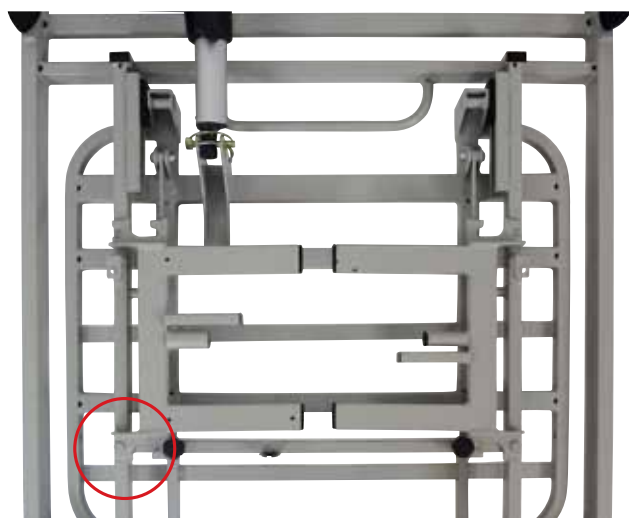

Regular maintenance

Visual inspection for obvious wear and tear of bracket. Replace immediately if you see evidence of tears, cracks or other general wear.

Care & cleaning

Hand clean the bracket using hand spray and wipe methods only. Novis recommends use of officially recognized disinfectants and mild cleaning solutions.


Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or the manufacturer for a list of available spare parts.

Specifications


CODE	BE4017
PRODUCT WEIGHT	2.4 kg per pair
CONFORMS TO	ARTG176883
MADE IN VIETNAM	

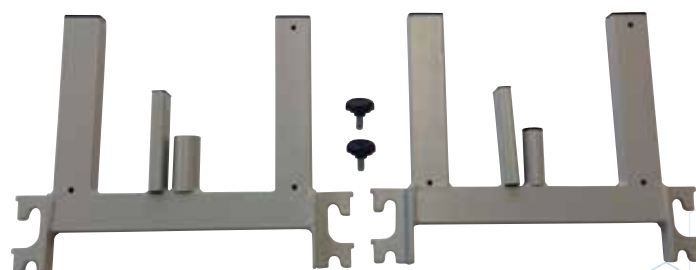

BetterLiving Community Bed – IV Pole

SET - UP AND OPERATION

The IV pole is designed to hold IV bags only. It is supplied assembled, with a left-hand and a right-hand fixing bracket (to attach to the bed).

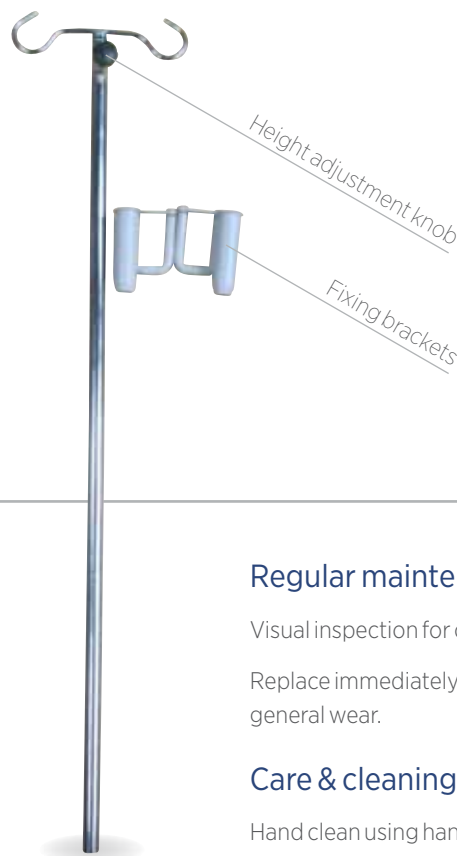

⚠ Please read these instructions before use.

⚠ This accessory is designed for use with the BetterLiving Community Bed only.

Only approved bed accessories listed in the BetterLiving Community Bed Parts List should be used with the bed unless otherwise specified by Novis Healthcare.

Attach bracket to the bed

Remove or move mattress to create space at the head of the bed frame.

- 1 Select either the left-hand or right-hand fixing bracket as needed to attach to the bed frame.
- 2 Loosen the screws with a Hexagonal Allen Key so the fixing plate can fit over the Bed frame.

⚠ Bracket is recommended to be placed on the inside of the bed frame (between the frame and the platform)

- 3 Position the bottom of the fixing bracket to the bed frame and tighten the two screws with a Hexagonal Allen Key.

⚠ Novis recommends installing both brackets (so as not to separate the accessories)

Insert the Pole

- 4 Insert IV the pole into the fixing bracket. Check the pole is securely held by the fixing bracket.

To remove the IV Pole, pull vertically upwards.



To adjust height

Pull the black height adjustable knob to release the locking mechanism to raise or lower the height of of pole as needed.



Regular maintenance

Visual inspection for obvious wear and tear of pole and components.

Replace immediately if you see evidence of tears, cracks or other general wear.

Care & cleaning

Hand clean using hand spray and wipe methods only. Novis recommends use of officially recognized disinfectants and mild cleaning solutions.

Warranty

Novis Healthcare Pty Ltd warrants this product for a period of 12 months. Please refer to our Warranty Statement for full details.

Please contact your authorised dealer or the manufacturer for a list of available spare parts.

Specifications

CODE	BE8003
PRODUCT WEIGHT	1 kg
HEIGHT	786 - 1430 mm
CONFORMS TO	ARTG176883
MADE IN VIETNAM	